

SDO[®] Lite

For other languages, please download from: www.jobskin.co.uk/wash

How to care for your new made-to-order garment

Before washing your garment, please follow these simple rules to help protect and prolong the life of your garment:

REMEMBER

- Wash your garment 2-3 times a week to rejuvenate the fabric
- Turn inside out and fasten zips and hook and loop tabs
- Wash dark colours separately
- Do not dry in direct sunlight, on a radiator or in a tumble dryer
- DO NOT USE A FABRIC CONDITIONER OR A DETERGENT THAT CONTAINS AN ADDED FABRIC CONDITIONER

HAND WASH

- If hand washing, roll your garment inside a thick towel to remove any excess moisture
- Rinse well and dry flat on a clean, dry towel. Do not wring

MACHINE WASH

- Use a gentle 30 wash cycle
- Use a non-biological mild detergent, DO NOT use fabric conditioner in any format
- Use a laundry bag if available



Hand Wash



Machine Wash at 30°C



Do Not Bleach



Do Not Iron



Do Not Dry Clean



Do Not Tumble Dry



Do Not Wring



Dry Flat



Drip Dry